Mammotome[®] EX Cover

Reference Guide


**INTENDED
USE**

- The cover is intended to be used as an instrument cover for medical devices and helps to prevent transfer of microorganisms, body fluids and particulate material.

WARNING

- *In the USA, federal law restricts device to sale or use by or on order of a physician.*
- *Before use, you should be trained in ultrasonography. For instructions on the use of your medical device, see your system's manual.*
- *Always use a cover to protect patients and users from cross-contamination.*
- *Disposable components are packaged sterile and are single-use only. Do not use if integrity of packaging is violated or if expiration date has passed.*
- *Inspect cover to ensure there are no holes or tears.*
- *Secure cover with enclosed bands.*
- *Do not reuse, reprocess or resterilize single-use device. Reuse, reprocessing or resterilization may create a risk of contamination of the device, cause patient infection or cross-infection.*

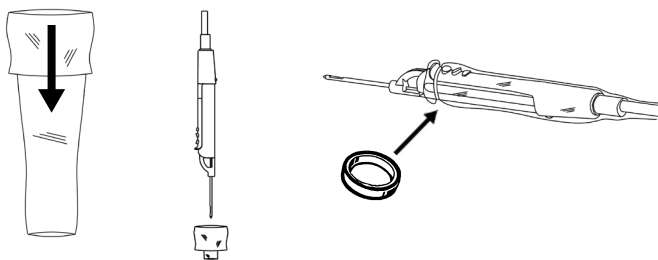USING COVER**DISPOSAL****WARNING**

- *Dispose of single-use components as infectious waste.*

CLEANING**WARNING**

- *Users of this product have an obligation and responsibility to provide the highest degree of infection control policies established by your facility.*
- *See your system's manual for reprocessing between uses.*

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COPYRIGHT ©2022 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW.
CIVCO Medical Instruments Co., Inc.